

Hugs[®] and Kisses[®] Approved Cleaning Materials

TECHNICAL BULLETIN





DESCRIPTION

This Technical Bulletin provides important information about which cleaning materials have been approved/not approved by Securitas Healthcare for use on Hugs Infant Protection and Kisses Mother-Infant Matching Tags.

Clients are advised to review the list of approved and non-approved cleaning materials carefully since manufacturers often provide a range of similar sounding product variants with different active ingredients. This may result in one variant of cleaning material being approved while another variant from the same manufacturer possibly not being approved. For example, Clorox Commercial Solutions® Germicidal Wipes are approved for use on Hugs and Kisses Tags whereas Clorox Healthcare™ H2O2 Cleaner disinfect wipes are not approved.

CLEANING INSTRUCTIONS

Hugs and Kisses Tags are not sterilized at the factory. They must be cleaned before first use and before inserting into tag charging station after patient use. In general, customers should consult with their Infection Control representative for cleaners available to the hospital that have been approved for use on plastics and are on the Approved Cleaning Materials list in this Technical Bulletin. Customers must follow all applicable label instructions for these products. Many of these products

customers must follow all applicable label instructions for these products. Many of these products specify a two-step process with specific surface contact times and/or instructions for rinsing away residue remaining on the tag.

<u>Note:</u> It is very important to rinse and thoroughly dry all tags after cleaning. If residue remains on the tag, the water evaporates away, leaving the chemicals to increase in concentration. A normally benign, mild alkaline solution then becomes a strong (and damaging) alkaline solution. This could cause serious degradation to a tag's plastic casing. Refer to the MSDS for cleaning specifications and dry times.

If required, a disposable soft-bristle brush may be used to remove surface dirt from the tag's surface and to gently clean inside the tag prongs.

For minimal surface degradation and cracking over time, use the following guidelines:

- Do not use pathogenic cleaners specified for tuberculosis (TB).
- Do not soak the tag.
- Do not use an autoclave to clean the tag or serious damage may result.



APPROVED CLEANING MATERIALS

The following list of cleaning materials has been approved by Securitas Healthcare for use on Hugs and Kisses Tags:

Cleaning Materials (Approved)

3M[™] HB Quat Disinfectant Cleaner Concentrate 25 3M[™] Neutral Quat Disinfectant Cleaner Concentrate

Brulin® Maxima 128

CaviCide™ or CaviWipes™

Clorox Commercial Solutions® Germicidal Wipes

Clorox® germicidal bleach (bleach use with gauze)

Coverage™ HBV

Ecolab® Airkem® A-456-N

Ecolab® Asepti-zyme™

Ecolab® Optipro™ Solid Multi Enzymatic Manual Cleaner

Expose® 256

Expose® II 256 or Phenolic 256 DC™

Formula C™

Frekaderm Actief 100 g liquide

Lingettes Prodene Aseptil (FDSUL233)

Metrex CaviWipes™

Metrex Detergezyme™ - generic detergent + enzyme

Oxivir® Five 16 Smart Dose

Precise® Hospital Foam Cleaner Disinfectant

Premoistened Alcohol/DI Clean Wipes

Professional Amphyl® Hospital Bulk Disinfectant

Prolystica® Ultra Concentrate Enzymatic Cleaner

Sani-Cloth® AF3 Germicidal Disposable Wipes

SaniWipes / Red Top = Sani-Cloth® Plus

Spor Klenz® - Ready To Use

Staph-Attack™

Thymo-Cide

Vesphene® II SE

Virkon® -S Powder

Virox™ Accel® Prevention (wipes & concentrate in use)

Virox™ Rescue™

NON-APPROVED CLEANING MATERIALS

The following list of cleaning materials has not been approved by Securitas Healthcare for use on Hugs and Kisses Tags. DO NOT USE these cleaning materials:

Cleaning Materials (Non-Approved)

Azul® Staphicide -Eco Line 0.5%

Bulk Disinfectant - not defined - many versions

Clorox Healthcare™ H2O2 Cleaner disinfect wipes

FDS Surfanios Premium NPC

Hibiclens®

Hydrogen Peroxide Cassettes

Isopropyl Alcohol use with gauze

LpH® Disinfectant Cleaner

Metrizyme®

Oxivir® TB/Oxivir® TB wipes

SaniWipes / Purple Top (Sani-Cloth®)

Sporicidin®

Virex® II 256

Wexcide 128

Wexcide-Ready-To-Use

4600 Vine Street Lincoln, NE 68503 +1888 622 6992

North America info@securitashealthcare.com

International global@securitashealthcare.com

About Securitas Healthcare

Securitas Healthcare empowers caregivers to deliver connected, productive and safe care. Its innovative portfolio of healthcare solutions helps over 15,000 hospitals, clinics and senior living organizations worldwide protect people, use assets efficiently and understand their operations for a caring and healing environment. Securitas Healthcare is proud to be part of Securitas, the world's leading intelligent protective services partner. For more information, visit us at securitashealthcare.com

Doc: 0970-090-000 REV G | Published: 2023/06/11 | KB Article 8184